Root Sectioning Kit Samarani 1

Developed with Dr. Rawad Samarani

NEW

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The Root Sectioning Kit Samarani 1 consists of 6 elevators and a small hammer. It was specially developed by Dr. Rawad Samarani for the safe and novel POP-OUT root extraction and is used in combination with Kit BRSS2 for root sectioning. The root fragments are removed from the socket by light, horizontal hammering using the newly designed elevators. The load is applied to the fragment by means of the sophisticated angulation of the elevator's working part, which is specifically adapted to the respective anatomical position where the extraction takes place. The laser markings on the elevators indicate the intraoral position of application (buccal, palatal / lingual). Additional elevators are included for special cases where the root to be extracted is below the level of the alveolar bone. The instruments can be conveniently sterilized directly in the tray supplied and transported safely. The elevators do not put loads on the adjacent tissue as it is the case with conventional extraction techniques. The innovative load application significantly reduces the risk of injury of soft tissue, bone or adjacent teeth. The high level of safety of this extraction technique is a decisive advantage, particularly when immediate implant placement is intended.

Anwendungsvideo
Application Video

Art.-No. BRSS1

© Dr. Rawad Samarani

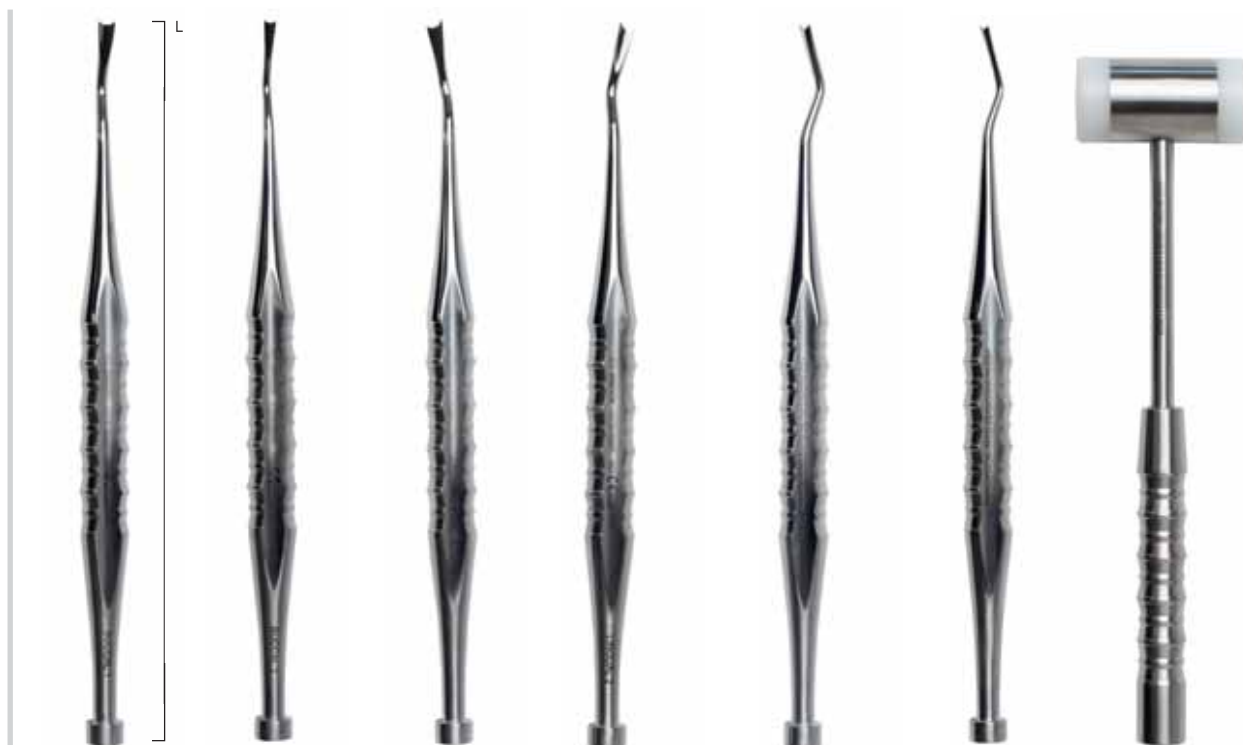
Fig.	MI201	MI202	MI203	MI204	MI205	MI206	MI155
Name	Luxationsinstrument Buccal 4,0 Luxation instrument Buccal 4,0	Luxationsinstrument Buccal 2,8 Luxation instrument Buccal 2,8	Luxationsinstrument Lingual, links gewinkelt Luxation instrument lingual, left angled	Luxationsinstrument Lingual, rechts gewinkelt Luxation instrument lingual, right angled	Luxationsinstrument 4,0 Luxation instrument 4,0	Luxationsinstrument 2,8 Luxation instrument 2,8	Hammer S Hammer S
Length mm	170.0	170.0	170.0	170.0	170.0	170.0	160.0